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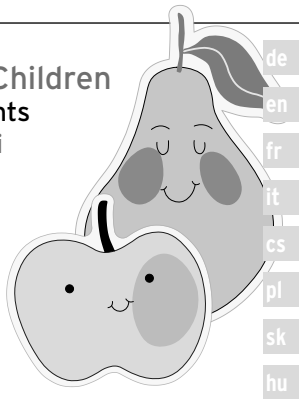
Dear Customer

Please read the instructions for use carefully and keep them for future reference.

en

Intended use and application

Cold & Warm Compresses for Children

Reusable medical product for external use in private households (Contains 2 compresses)

Do not apply the compresses directly to the surface of the skin - always use a protective cloth. Test the temperature of the compress on the inside of your wrist before using it.

1. For application as a cooling treatment...

... to treat bruises, sprains, strains, toothache, headaches, insect bites or other types of inflammation.

The max. application time for use as a cooling treatment is 5 minutes; this may be repeated at intervals of 5 minutes.

For the cooling treatment, place the compresses in the freezer for at least 1 hour at a minimum temperature of -15 °C.

2. For application as a warming treatment...

... for non-inflammatory cases, such as muscle tension or stomach cramps.

To heat up the compresses...

... place them in a saucepan of water at a temperature of max. +80 °C for approx. 3-5 min (do not boil them).

... place them in a bowl of water and heat them in the micro-

wave at max. 600 W for approx. 3-4 min.

The compresses must not be allowed to touch the bottom of the saucepan while being heated. Do not take the compresses out of the water with your bare hands - always use kitchen tongs.

en

Safety warnings and precautions

The compresses...

... are designed for external use only and are to be used solely for their intended purpose. The gel is not suitable for consumption.

... must not be used on open wounds or broken skin.

... must be kept out of the reach of children and only used under adult supervision.

... are not suitable for children under the age of 3.

... are not a toy.

... must not be used if they are damaged. If the gel comes into contact with skin, rinse the gel off under running water.

... must be removed from the skin immediately if the temperature begins to cause discomfort. Be aware that small children are more sensitive to heat and cold than adults.

... must not be used in combination with pain medication or other pain treatments, or by individuals with reduced sensitivity to temperatures, for example in the case of arterial circulation disorders or Raynaud's syndrome.

... should only be used by people with rheumatic conditions, circulatory disorders or other medical conditions following consultation with a doctor.

... must always be wrapped in a cloth before use. Never use the compresses on bare skin. Risk of burns or frostbite.

Please note:

- The temperature of the compress must not be too low if it is used to cool parts of the head. Do not apply cold compresses for longer than approx. 5 minutes at a time. Risk of hypothermia!
- **Do not use the compresses as a warming treatment in the case of inflammatory conditions!**
- **Depending on the duration of application, even low temperatures can result in burns to the skin!**

- **Children must never bite or chew on the compresses! Do not use the compresses as a treatment for teething in infants!** The gel in the compresses contains a bitter substance to ensure that children immediately spit it out and do not swallow the gel if they do chew on the compresses. **In the event that the contents of the compress are swallowed accidentally, drink copious amounts of water and consult your doctor as a precaution.**
- **Overheating the compresses can damage the material of the pouches!**

Cleaning / storage

- Do not use any solvent products to clean the compresses (e.g. disinfectant solution).
- Store the compresses at room temperature in a location protected from direct sunlight. The volume of the compresses reduces over time. Replace the compresses, if necessary.
- Dispose of the compresses along with standard household waste.

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PS Product Services GmbH,