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## EN Heat Patch for Neck Pain

Deep-acting and relaxing heat therapy for the temporary relief of neck pain as well as muscle and joint pain.

Ingredients: Iron, Water, Diatomaceous Earth, Activated Carbon, Sodium Chloride

### Usage

- » The product is activated by contact with air, so open the airtight pouch just before use using your hands (do not use a knife or scissors).
- » Remove the protective film from the back of the product and apply it firmly and smoothly to the painful area with the adhesive side.
- » It may take up to 30 minutes to reach the full temperature.
- » For maximum effectiveness, we recommend a maximum application time of 12 hours per day.
- » The product is for single use only and should be disposed of in household waste after use.
- » Temperature development and duration vary depending on the environment. The heating effect may be reduced if there is insufficient oxygen available.

### Contraindications

Heat patches should not be used by individuals with reduced sensitivity to temperature, such as those with circulatory disorders, diabetes, nerve damage, paralysis, or sensitive skin. Do not use on children under 12 years of age.

### Warnings/Precautions

- » This product can cause burns.
- » Keep out of reach of children, infants, and pets.
- » This product may cause itching, redness, or mild skin burns in people with sensitive skin or if used incorrectly.
- » Do not use while sleeping.
- » Check your skin regularly (approximately every hour) during use.
- » Consult a doctor before use if you have diabetes, poor circulation, heart conditions, circulatory disorders, rheumatic

diseases, nerve damage, skin disorders, rheumatoid arthritis, or if you are pregnant.

- » The powder inside the patch contains iron and may be harmful if it comes into contact with skin or is swallowed. In case of accidental ingestion, rinse your mouth with water and contact a poison control center or doctor.
- » If the contents of the heat patch come into contact with skin or eyes, rinse the area immediately with water.
- » Never heat the product in a microwave or attempt to reheat it, as the patch may catch fire.

### When using this product

If you feel unwell, find the temperature uncomfortable, or notice irritation, rash, or burns, remove the product immediately. If the patch feels too hot, stop use and/or apply it over clothing. Do not apply additional pressure to the product or wear tight clothing over it. Ensure that air can always circulate around the patch. Remove the product before undergoing an MRI scan.

### Do not use the patch if

- » the patch is leaking and/or the packaging is damaged or torn.
- » for more than 12 hours in a 24-hour period.
- » in combination with pain-relieving ointments, tablets, medicinal lotions, creams, or other salves.
- » on red, diseased, damaged, or injured skin.
- » on bruises or swellings that have occurred within the last 48 hours or on inflamed areas.
- » on parts of the body where you cannot perceive heat or are excessively sweating.
- » if you have arterial circulatory disorders or Raynaud's syndrome.
- » simultaneously with other heat-producing products.
- » by individuals who cannot remove the product themselves.
- » by individuals who are unable to follow the instructions for use.

### Stop use and consult a doctor if

- » your symptoms worsen or persist for more than seven days.
- » you experience pain, burning, swelling, rash, or any other changes on the skin where the patch was applied.

### Transport and Storage

The product can be stored and transported at room temperature (-10°C to +40°C) in a dry and ventilated environment.

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Koolcare Technology Co., Ltd
Prolinx GmbH
Thermopad GmbH