2018-12

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The tinted lenses of your new spare reading glasses are made of lightweight plastic.

Tip: You have chosen the right refractive power if you can clearly read small print held at a normal reading distance of 30-40 cm. In most cases, this is achieved with a refractive power of between +1.00 and +3.00 dioptries.

Warning notices:

- Only regular eye examinations carried out by an eyecare professional can determine your visual needs and eye health.
- This is a medical device.
Possible side effects: Use over long periods of time and differing pupil distance and dioptries can cause dizziness, discomfort and/or headaches.
- For near vision and reading use only.
Not for distance vision.
-  Not for driving or vehicle operation.

- Filter category 3: dark standard filter for medium and strong sunlight;
Visible light transmission: 8%-18%.
- The plastic lenses ensure optimum protection against harmful UV rays.

Warning notices:

- Not for protection against artificial light sources, e.g. solaria.
 - Not for looking directly at the sun.
 - Not for use as eye protection.
Not for use as eye protection against mechanical impact hazards.
 -  Not suitable for driving at night or under condition of dull light.
 - To protect the lenses from scratches, keep the glasses safe in the case.
- ▷ Clean the glasses with mild soapy water and wipe them dry with a soft cloth.
- ▷ Dispose of the glasses along with household waste.

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