Hallux-Valgus-Druckschutz

Hallux-Valgus Splint

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MD

Medizinprodukt / Medical device / Produit medical
Dispositivo medico / Zdravotnický prostředek
Wyrób medyczny / Zdravotnícka pomôcka
Gyógyászati termék
TCM2021 FP

LOT

TCM-LAN: 609932 / PS-LAN: FP0013

2023-03-01

PS Product Services GmbH,
Alte Heerstraße 6, 53757 St. Augustin, Germany

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en

Dear Customer

Read these instructions carefully and keep them for future reference. If you give this product to someone else, remember to also include these instructions.

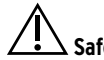
Intended use/Indication

The hallux-valgus splint is made using a skin-friendly thermoplastic elastomer. It provides pressure and pain relief in the case of bunions. Mechanical friction is reduced and the slanted position of the toes is gently corrected while the product is being worn. It must only be used for its intended purpose and is intended for personal use only. **Consult your doctor if you have any questions.**

This pair of splints for your left and right foot is suitable for all standard shoe sizes and can be easily worn under socks/tights. Both splints are washable and reusable.

Contra-indication

In the case of diabetes, circulatory or skin disorders, only use the product after consulting your doctor.

Safety warnings

- Keep the product and packaging materials out of the reach of infants. There is a risk of suffocation!
- If you experience redness, inflammation or persistent discomfort when wearing the product, remove it immediately. It must not restrict the flow of blood. The product must not be worn while asleep or by anyone who is unable to indicate discomfort.
- Only use the product on unbroken, healthy skin. The skin must be free of oils, creams, ointments, etc.
- Serious incidents which occur while wearing the product must be reported to the manufacturer and the relevant authorities.
- **The product must be applied/positioned as shown in the illustration (at the top of these instructions).** The toe spacer separates the big toe from the adjacent toe.

Care and storage

- ▷ Wash the product under running water before using it for the first time and after each use and allow it to dry at room temperature. Do not place it on a radiator to dry it!
- ▷ During normal use, the product releases very small amounts of oil. You may apply baby powder to the affected areas, as required.
- ▷ Store the product so that it is protected from dust, moisture, direct sunlight and extreme temperatures. Ensure that it is also protected from sharp or pointed objects.

Disposal

- ▷ Dispose of the product and packaging in accordance with current environmental regulations.

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